# L'espace

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**Intended use:**  
L'espace is intended for use by patients under the supervision of a doctor. The device is a valved spacer chamber indicated for the use of metered dose inhalers (MDI) and can be used by both children and adults at home or in the hospital. The device allows to improve the efficacy of drug administration at lung level by reducing the side effects.

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with mouthpiece  
with mask 0 - 2 years  
with mask 2 - 6 years  
with adult mask

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If the device is always used by the same person, clean it as shown.

**Disinfection:** after cleaning, soak the parts in a disinfectant solution (e.g. a solution containing sodium hypochlorite in concentrations not greater than 2%). Follow the instructions of the manufacturer of the disinfectant solution, and make sure it is compatible with the materials of the device. Rinse thoroughly with clean water.

If the device is used by more than one person, besides cleaning after each use, it needs to be sterilized in an autoclave at 121°C for at least 15 min. (preferably no longer than 30 min.). It can be sterilized without affecting the functional features up to a maximum of 20 times, after which it must be replaced. If needed, for a deep cleaning, it is possible to boil the device or to wash it in the dishwasher.

#### Important:

- The device should be replaced if any of the components gets damaged.
- Some components are small enough to be swallowed by children, so they must never be left unattended. An adult should always be present.
- Do not disassemble the device beyond what is recommended in the Cleaning Instructions.
- If you are not sure on how to use the device, talk to your doctor or pharmacist.
- To ensure efficient performance, the device must only be cleaned as instructed here.
- Follow the doctor's instructions regarding the type of drug and its dosage. Read the instructions of the drug before use.

#### Notes:

- Store in a clean container when not in use.
- The device should be replaced after 24 months of continual use.
- This product is latex-free.
- There are no contraindications to the use of the device.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

#### Storage conditions:

- temperature: -25°C to +50°C
- relative humidity: 30% to 95%
- atmospheric pressure: 700 hPa to 1060hPa

#### Disposal:

The device and/or its parts should be disposed of in compliance with current regulations and environmental protection laws. Where there are no legal requirements, we recommend recycling.

The table lists the materials of the various components of the device. All materials used are phthalate free and the presence of natural rubber has not been detected.

COMPONENT	MATERIAL
Backpiece	Silicone
Chamber	Polycarbonate
Mouthpiece	Polycarbonate
Valves	Silicone
Mask (if present)	Silicone
Tissue bag	80% cotton 20% polyester
Plastic bag	LDPE
Instructions	Paper
Box	Carton

#### Technical characteristics:

- Chamber volume: 220 cc (ml)
- Dead space\*:
  - o L'Espace Infant orange with mask: 34 ml
  - o L'Espace Pediatric yellow with mask: 49 ml
  - o L'Espace Adult magenta with mask: 88 ml

\* = Dead space measured following the procedure used in clinical test "Shah SA, Berlinski A and Rubin BK. Force-dependent Static Dead Space of Facemasks Used with Holding Chambers. Respir Care. 2006;51(2): 140-144"

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