Avvertenze

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Volume: 400 ml
Materiale: Polipropilene

Warnings

1. Read the instructions of the medicine before use
2. The device has no contraindications of use
3. If a severe accident directly connected to the use of the device occurs, it shall be immediately reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established
4. The reuse of the product may lead to risks of infection borne by the patient and the user
5. If you have any questions about how to use the product, consult your doctor or pharmacist
6. Before use, make sure that there are no foreign objects inside the valved holding spacer chamber
7. Replace the product if you notice that there are missing or incorrectly assembled parts
8. Do not leave the device unattended in the presence of children
9. Administer the dose of medicine prescribed by the attending physician
10. The device is latex-free
11. Do not remove the valved holding spacer chamber for any reason
12. After use, the device must be disposed of in the appropriate disposal bins
13. The manufacturer shall ensure safety, reliability and performance of the device, if the latter is used in accordance with the instructions for use laid down in this document
14. Any liability of the manufacturer and the warranty shall lapse if the user fails to comply with the provisions given in this document

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Environmental storage conditions	
Temperature range	-25 °C to 50 °C
Humidity	30 % to 95 %
Pressure	700 hPa to 1060 hPa

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- **Intended use**
Libero is intended for use by patients under the supervision of a doctor.
The device is a valved chamber indicated for the use of metered dose inhalers (MDI) and can be used by both children and adults at home or in the hospital.
The device allows to improve the efficacy of drug administration at lung level by reducing the side effects.

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