Fabbricante / Manufacturer:
Medinet S.r.l. - Via Canova 19/a
20145 Milano - Italia
www.medinetsrl.com

IT RESPIPROGRAM

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EN RESPIPROGRAM

Product description

Respirogram device is a flow oriented respiration exerciser designed to encourage deep inspiration (inhale). Deep breath helps to expand the lung alveoli as well as to clear the airway passage of mucus.

Indications

To prevent postoperative respiratory complications, eg. atelectats, pneumonia or pulmonary dysfunction.

Contraindications

For eventual contraindications, address to your doctor or therapist.

Contents of the bag

A) Respirogram; B) Insertion point with filter; C) Tube with mouthpiece; D) Detail of the tube insertion; E) Seat for mouthpiece with tube

Directions for use

Remove the device from package and connect the filter end of tubing to the exerciser. The nurse may label the exerciser with your name in the white base of the device to avoid the inadvertent use by another patient.

Seat comfortably to promote optimal lung expansion. If you are unable to assume or maintain this position, perform the procedure in any position as long as the device remains upright. Tilting the respiration exerciser decreases the required effort and reduces the exercise's effectiveness. Exhale normally and place lips tightly around mouthpiece.

- 1) INHALE maximally trying to raise one, two or three balls according the instructions received from your doctor or nurse.
- 2) HOLD breath for 2 to 3 seconds and
- 3) EXHALE slowly through the nose, while removing the mouthpiece from lips
- 4) Relax after each deep breath and breathe normally.
- 5) Repeat the process at the frequency prescribed by your doctor or nurse.
- 6) Cough after the exercise to remove accumulated secretions. You may use a small pillow for splinting chest incisions.

Caution

Use this device only under the prescription and directions given by your doctor or nurse. If you feel light-headedness during the procedure, stop and rest before continuing. In paediatric use, the child must be supervised during exercise. As any medical device, keep it out of the reach of children.

Cleaning the Respirogram

The Respirogram is a "single patient" device and should be used during the period prescribed by your doctor. After use wash carefully the mouthpiece with clear warm (approx. 40°C) spin water and leave it dry. You may place the tube with the mouthpiece in its seat at the right of the body, the mouthpiece facing down. If necessary, Respirogram can be dismantled opening the white base, allowing the separate washing of the body and the balls. When assembling the device back, once the pieces completely dry, be sure to place each ball in its corresponding cylinder: The red in the 600cc marked cylinder, the yellow in the 900cc and the green in the 1200cc. When not in use, keep the device in a dry clean place. The device is latex free and may be disposed in the plastic clearance container.

Notes:

The Respirogram is not a flow measuring device. The flow figures shown in each cylinder are only given as indication for the doctor or nurse. The filter located in the insertion point of the tubing serves only to prevent inhalation of extraneous material. Notice to the user and/or patient: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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