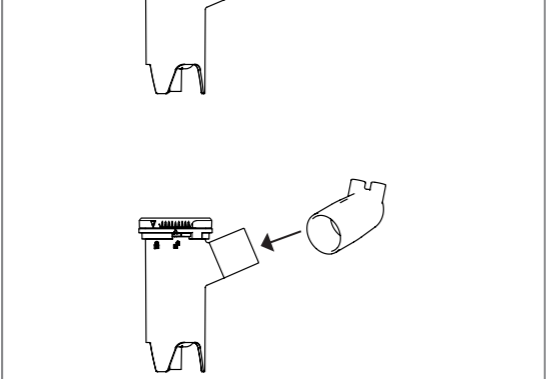
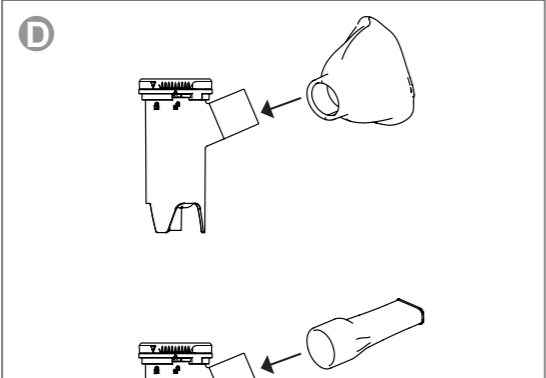
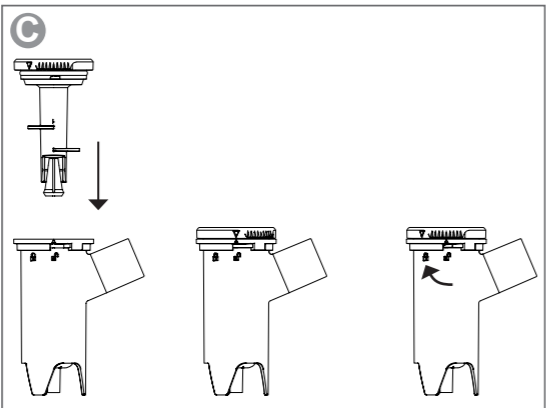
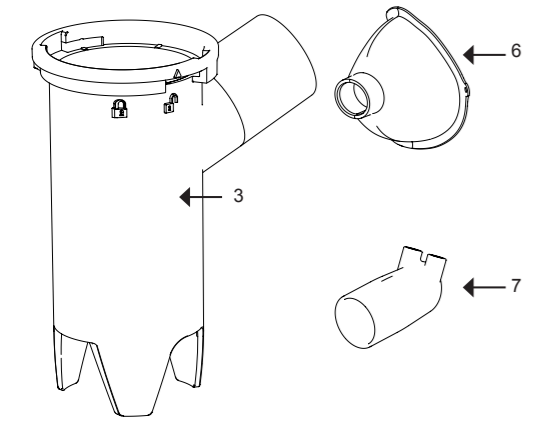
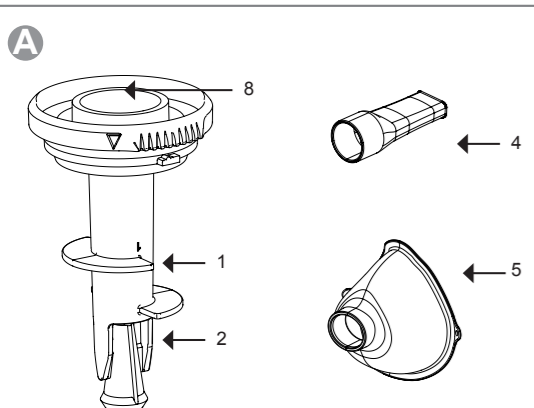
SoCu H Junior set
SoCu H Basic
SoCu H Adult set
SoCu H

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⚠ Please read the instructions for use and the important notes (section 1.4) carefully before using the device.

1 - IMPORTANT INDICATIONS

1.1 - INTENDED USE

This medical device falls under the category of "aerosol therapy accessories" and is intended for use in the administration of a solution in aerosol form.

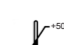
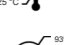
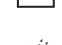
1.2 - INDICATIONS FOR USE

The device is a nebuliser intended to be used for the administration of drugs in aerosol form. The nebuliser must be connected to a pneumatic aerosol therapy device that generates compressed air. It can be used to treat diseases of the lower airways (including bronchial asthma, chronic bronchitis, pulmonary emphysema and cystic fibrosis).

⚠ The administration of drugs requires a medical prescription specifying the type of drug, the dosage and the length of treatment.

SoCuh H is designed and made in Italy in accordance with EU Regulation 2017/745 on Medical Devices (as amended and updated) and bears the CE marking.

1.3 - SYMBOLS USED

	Important, read the instructions carefully		Attention!
	Lot number		Product code
	UDI code		Medical device
	Temperature limits		Production date
	Relative humidity limits		Keep the device dry
	Pressure limits		Manufacturer
	Does not contain phthalates		No traces of natural latex
	CE mark		Do not dispose of in the environment

1.4 - IMPORTANT NOTES

- Read the instructions carefully and keep them in a safe place for future reference.
- The nebuliser must be connected to a source of compressed air with a pressure between 1.5 bar and 3.2 bar.

- The device must only be used for treating the lower airways in accordance with these instructions for use. It must not be used for any other purpose.
- There are no contraindications to the use of the device.
- Some components are small enough to swallow and may lead to suffocation.
- Read the list of components in the table in section 7 to see if there are any components that have caused an allergic reaction in the past.
- The material of the nebuliser might not be compatible with solutions that have not been evaluated.
- Consult the manufacturer's instructions of the device with which the NEBULISER is used, to make sure they are compatible (see 6 - TECHNICAL DATA).
- Children must only be allowed to use the device under the supervision of an adult who has read and understands these instructions.
- The administration of drugs requires a medical prescription specifying the type of drug, the dosage and the length of treatment
- Do not use the nebuliser in the bath or shower.
- The manufacturer has defined a service life of at least 5 years from first use.
- Should a serious accident occur in connection with the device, the manufacturer and the competent authority of the Member State in which the user and/or patient is established must be notified.
- In case of doubt about the use of the product, contact the manufacturer or physician or pharmacist.

1.5 - PATIENT GROUPS AND ENVIRONMENTS OF USE

The nebuliser provides highly effective inhalation therapy for patients of all ages, from small children to adults, and can be used in both home and hospital settings.

1.6 - DESCRIPTION OF THE DEVICE

The SoCu H is a nebuliser specifically for treatment of the lower airways. The nebuliser must be connected either to a pneumatic aerosol therapy device or to a source that generates compressed air. The SoCu H nebuliser is of the double-Venturi type, as it has an opening on the cover that allows very fast nebulisation. The SoCu H nebuliser may be connected to an antibacterial filter with 22F connector in accordance with Standard ISO 5356-1.

The SoCu H nebuliser is available in four versions:

- SoCu H: with mouthpiece, nosepiece, connecting tube and connector
- SoCu H Basic: with mouthpiece, connecting tube and connector
- SoCu H Junior set: with paediatric mask with elastic band, mouthpiece, connecting tube and connector
- SoCu H Adult set: with adult mask with elastic band, mouthpiece, connecting tube and connector

1.7 - CONTENTS

The package contains (Fig. A) the parts listed below. A nebuliser comprising the following:

- upper section (1) with atomiser (2) and connector for filter (8)
- lower section (3)

A patient interface device, which depending on the version chosen may be:

- mouthpiece (4)
- adult mask (5), with elastic band for attachment to the face
- paediatric mask (6), with elastic band for attachment to the face
- nosepiece (7)
- connecting tube and connector

The lower section (3) is fitted with a terminal to which the air supply tube is connected.

2 - PREPARATION AND USE

- ⚠

SoCu H requires connection to a source of compressed air. In all cases, it is important to make sure that the source of air is compatible with the nebuliser (see section 6 - TECHNICAL DATA).

Please read section 3 before using the device.

Proceed as follows to use the SoCu H:

- the lower section of the nebuliser (3) has a graduated scale that shows the approximate amount of liquid inside. The purpose is merely to give an approximate indication of the contents. Use a graduated syringe to get an accurate dose. Pour the liquid into the lower section of the nebuliser (Fig. B) according to the amount recommended by your physician
- place the upper section of the nebuliser (1) on the lower section (3), positioning the triangle at the open padlock and turn the cover until the triangle is aligned with the closed padlock (Fig. C).
- attach the mask (5) (6), mouthpiece (4) or nosepiece (7) to the lower section of the nebuliser (3) (Fig. D)
- connect the lower end of the tube to the nebuliser and the other end to the air intake of the aerosol device
- start up the device
- during treatment, hold the nebuliser upright in your hand to enable it to work efficiently
- if the above operations have been performed correctly, a spray will come out of the mask/mouthpiece.

The mask (5)(6) must be placed gently on the patient's face so that the nose and mouth are covered. The mouthpiece (4), on the other hand, must be placed between the lips: for the therapy to be effective, the user must breathe in through the mouth and out through the nose. When you have used up all the drug or interrupt treatment, switch off the aerosol device or turn off the air supply. A small amount of liquid may remain in the nebuliser at the end of the session but this has no importance. After use, disconnect the nebuliser from the tube and the device from the mains.

3 - CLEANING, DISINFECTING AND STERILISING

- ⚠

Before proceeding, unplug the device and disconnect the nebuliser and tube from the device.

- ⚠

If it is necessary to clean, disinfect or sterilise this medical device, using a method other than that given in this section, contact the manufacturer first for a copy of the data sheet.

To prevent the risk of microbial contamination, SoCu H must be cleaned and disinfected, and if necessary sterilised, after each treatment and before use, following the instructions carefully. Before cleaning, disinfecting or sterilising any of the components, disassemble them as follows:

- remove the tube
- remove the mask, mouthpiece or nosepiece (4, 5, 6 or 7) from the lower section of the nebuliser (3)
- detach the upper section (1) from the lower section of the nebuliser (3)

3.1 - CLEANING

The disassembled nebuliser, the masks and the mouthpiece can be washed in water at an approximate temperature of 40°C with the addition of detergent (refer to the manufacturer's instructions), then rinsed in running water. When all the parts have been cleaned, dry them with a soft cloth and place in the storage compartment.

3.2 - DISINFECTING

After washing, all the components of the nebuliser, the mouthpiece and the nosepiece can be immersed in a cold disinfectant solution (e.g. a solution containing up to 2% sodium hypochlorite). Follow the manufacturer's instructions. The components of the SoCu H nebuliser, with the exception of the masks and the connecting tube, can be disinfected by boiling in a saucepan of water for a maximum of 10 minutes. The components must not be in direct contact with the bottom of the pan. The boiling cycle may alter the colour or appearance of the components; this does not affect the device.

3.3 - STERILISING

The components of the nebuliser (with the exception of the masks and the connecting tube), the mouthpiece and the nosepiece can be sterilised in an autoclave at 121°C for at least 18 minutes, for up to 20 times. The PVC masks cannot be sterilised in an autoclave.

- ⚠

Sterilisation is recommended when the device is used in a hospital setting or by more than one patient.

The sterilisation methods have been validated and documented by the manufacturer in Technical Test S-4/12 no. 317. Repeated sterilisation cycles do not affect the quality and performance of the product. The manufacturer does not guarantee the sterile state of the device

4 - MAINTENANCE

When cleaning, disinfecting or sterilising SoCu H components, you can check them visually and determine when they need to be replaced. Repeated sterilisation may affect the materials of which the nebuliser is made, so the components should be inspected regularly. The SoCu H nebuliser must be replaced when there are clear signs of deterioration. In any case, it is recommended to replace the nebuliser at least once a year or, in the case of frequent use, to replace it when its characteristics change.

5 - PROBLEMS, POSSIBLE CAUSES AND SOLUTIONS


PROBLEMS	POSSIBLE CAUSES	SOLUTIONS
The nebuliser does not generate a spray.	- The device is switched off.	- Switch on the device.
	- The liquid has not been poured into the nebuliser.	- Pour liquid into the nebuliser.
	- Too much liquid has been poured into the nebuliser.	- Reduce the quantity of drug in the nebuliser to below the maximum level.
	- The nebuliser has not been assembled correctly.	Check that the upper section of the nebuliser (1) is properly inserted and secured in the lower section of the nebuliser (3)
	- The air outlet in the base (3) is blocked.	- Unblock the hole.
	- The air tube is not connected correctly	- Connect the air tube properly
The nebuliser does not perform as stated.	- The air source does not provide the required pressure and rate of flow	- Check the characteristics of the air source
The drug does not reach the mouth or there is difficulty breathing	- The mask (5 or 6) or mouthpiece have not been fitted correctly	- Assemble the spacer mask (4, 5) correctly.

6 - TECHNICAL DATA

Minimum supply pressure	1,5 bar	
Maximum supply pressure	3,2 bar	
Air flow at minimum pressure	7 l/min	
Air flow at maximum pressure	12 l/min	
Maximum nebuliser filling volume	9 ml	
Recommended filling volume	3 ml	
MMAD, Diagram 1 (*) (**)	4.33 µm	
GSD (*) (**)	1.98 µm	
Breathable fraction (%<5 µm) (*)(**)	57.52%	
Particle distribution (*)(**)		
%>5 µm	42.48%	
2 µm <%< 5 µm	40.69%	
%< 2 µm	16.82%	
Residual volume	1,14 ml	
	At minimum pressure and flow	At maximum pressure and flow
Aerosol output	2,02 ml	2,00 ml
Aerosol output rate	0.301 ml/min	0.372 ml/min
Percentage of filling volume per minute	10.03%	12.40%

() The following information on nebuliser performance is based on tests using adult ventilator schemes and is likely to differ from those claimed for paediatric or infant populations. The use of a high-viscosity suspension and/or solution may alter the particle size distribution curve, the MMAD, aerosol emission and/or aerosol emission rate, which may therefore differ from those disclosed by the manufacturer. Tests performed with 0.9% NaCl saline solution*

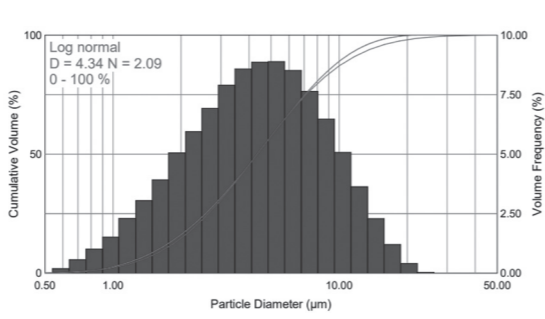*(**) The MMAD values shown are measured with the Malvern Spraytec instrument and using NaCl 0.9% saline.*

7 - DISPOSAL

If taken out of use, the device and/or its components must be disposed of in accordance with the regulations in force and environmental protection.

The table shows the different types of device components. Check the regulations in force in your municipality. None of the materials used contains phthalates or traces of natural Latex.

COMPONENT	MATERIAL
Upper section (1)	Polypropylene
Lower section (3)	Polypropylene
Mouthpiece (4)	Polypropylene
Adult mask (5)	PVC
Paediatric mask (6)	PVC
Nosepiece (7)	Polypropylene
Connecting tube	PVC
Connector	Nylon

8 - OPERATING CONDITIONS

- temperature from +5 °C to +40 °C
- relative humidity from 15% to 93%
- atmospheric pressure from 700 hPa to 1060 hPa

9 - STORAGE CONDITIONS

- temperature from -25 °C to +50 °C
- relative humidity less than 93%
- atmospheric pressure from 700 hPa to 1060 hPa

10 - WARRANTY CONDITIONS

The manufacturer is liable for the safety, reliability and performance of the device, provided that it is used in accordance with the instructions and for the intended use only, and any repairs are carried out by the manufacturer or an approved service centre. The manufacturer declines all liability and the warranty will be invalidated if the user fails to follow the instructions and the important notes contained herein.

11 - TECHNICAL FEATURES UPDATE

Air Liquide Medical Systems S.r.l. periodically reviews and modifies all its medical devices in order to improve their performance, safety and reliability. The instruction booklets are updated accordingly to include any new or changed features. If the booklet accompanying the device gets damaged or mislaid, a replacement copy can be obtained from the manufacturer by quoting the data shown on the rating plate.

12 - COPYRIGHT

The information contained in this booklet must not be used for any purpose other than the reason for which it is provided. This booklet is the property of Air Liquide Medical Systems S.r.l. and no part of it may be reproduced without the manufacturer's written permission. All rights reserved.

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